


How to download the HFX App

1. Click your App Store icon. 
2. Tap the **Search** icon on the bottom right side of your screen.
3. In the search bar type, "**HFX**" and then tap **Search**.
4. Tap on the **HFX App** and tap **GET** to download the app.


You will need to know your existing Apple ID and password to download the HFX App from the Apple App Store.



Or scan this code to go to the Apple App Store.


Get familiar with your HFX App Home screen

1. Stim ON/OFF Button
2. Current Program Setting
3. Change Program Button
4. Start Assessment
5. Battery Status
6. Bluetooth Connection Status
7. My Progress Tab
8. Favourites Tab
9. Menu Tab

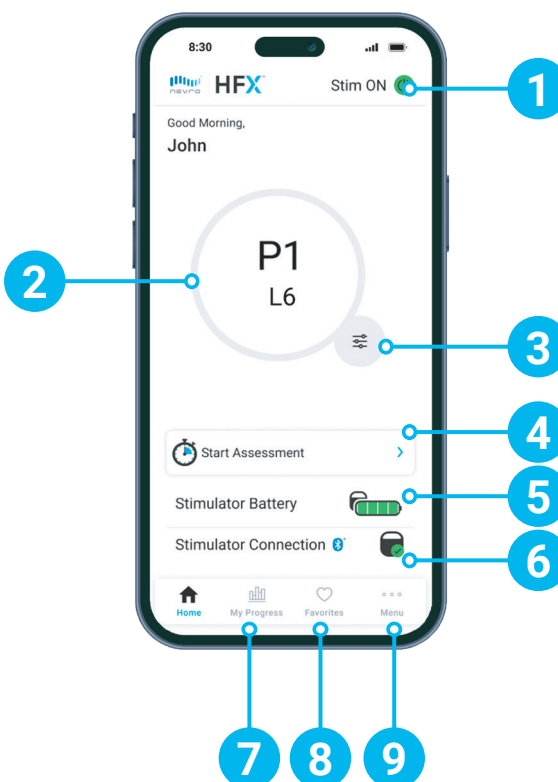

How to change programs

1. Tap the Change Program Button on the lower right of the circle.
2. To change your program setting, scroll through the program carousel until the circle is over the program you want.
3. To change your level setting, scroll through each level on the level wheel until the level you want is selected.
4. Tap Done to save your changes.

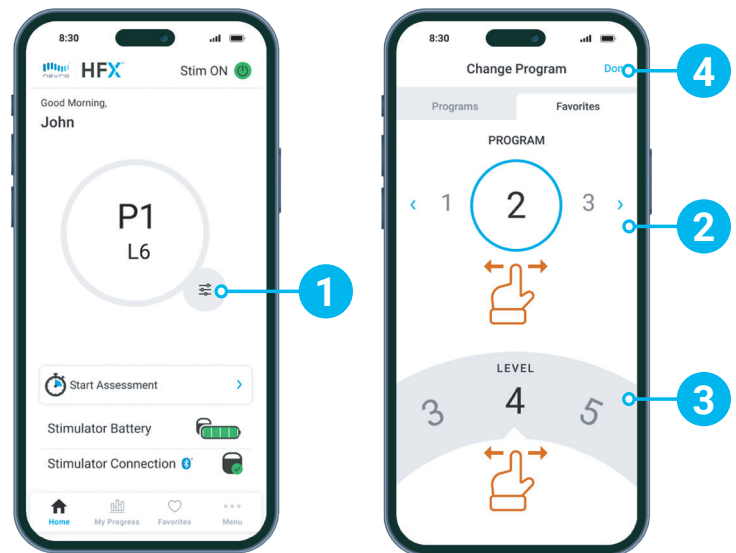

Trial guidelines

- Keep your mobile phone within close distance to your trial device because they will communicate.
- You will work closely with your Nevro Care Team to adjust your program settings as needed.
- Check the Stim ON/OFF Button on your HFX App twice a day to confirm your trial device is ON.

How to complete an assessment

- When you're within your assessment window, the Start Assessment button will turn yellow.



Your Nevro Care Team Contact Information

+61 488 845 285

Monday - Friday 9:00am - 5:00pm



Nevro Medical Pty Ltd
Level 14, 440 Collins Street
Melbourne VIC 3000, Australia
+61 488 845 285

HFX iQ, the HFX iQ logo, NEVRO, and the NEVRO logo are trademarks or registered trademarks of Nevro Corp.
© 2025 Nevro Corp. All Rights Reserved. S005298